

Is PROKERA® comfortable?



PROKERA® is similar to a large contact lens. You may experience a foreign body sensation that is uncomfortable, but it should not hurt.

To aid healing and reduce your eye's exposure to outside dryness, your eye doctor may partially close your eyelid after PROKERA® is inserted.

Are there any special instructions to follow while PROKERA® is inserted?

- Avoid rubbing your eyes, strong blinking, or moving PROKERA® with your fingers
- Do not remove PROKERA® without consulting your eye doctor first
- Do not swim or soak your face with water
- Shower only when the eye is tightly closed
- Do not drive or operate heavy machinery or perform functions that require unobstructed vision or good depth perception
- Use eye drops and other medications as prescribed by your eye doctor
- Contact your eye doctor right away if you are uncomfortable or have any other problems with PROKERA®

Reduce inflammation and promoting healing for a healthy ocular surface

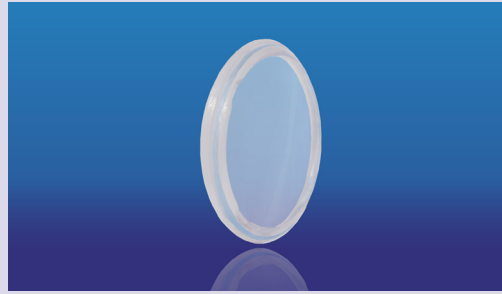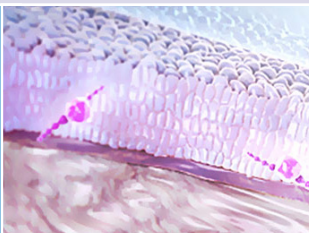

About PROKERA®:

PROKERA® is a therapeutic device that simultaneously reduces inflammation on the surface of your eyes and promotes scar-less healing of your cornea.

PROKERA® allows easy insertion and removal of the devices in your doctor's office. When PROKERA® is placed on your eye, it will aid in healing, minimize scarring, and decrease inflammation and related pain.

Ask your eye doctor for more details about using PROKERA® to treat your eye surface condition.




Provided by Bio-Tissue,
Manufacturer of PROKERA®
www.biotissue.com

SM-043 - REV D 02/16/16

Patient's Guide To:

PROKERA®

*Reducing inflammation
and promoting healing for
a healthy ocular surface*

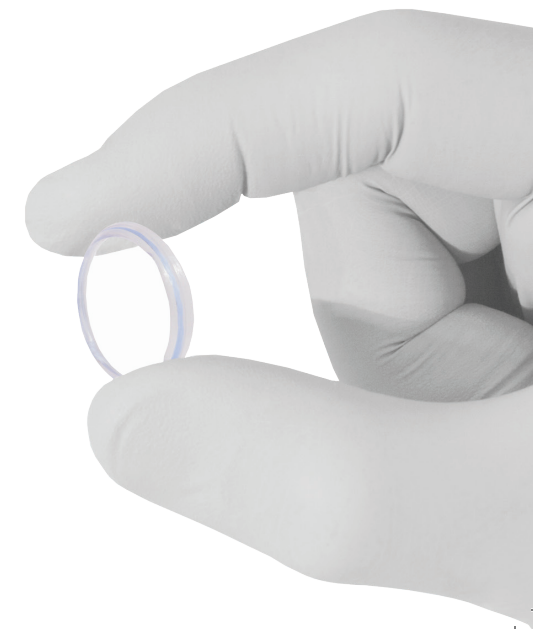

What is PROKERA®?

PROKERA® is a therapeutic device used by eye doctors around the world to protect, repair and heal damaged eye surfaces. PROKERA® is made by fastening a piece of amniotic membrane to a polycarbonate ring.



PROKERA®
We've Got You Covered

Where does the amniotic membrane come from?

The placentas used to prepare PROKERA® are donated by consenting mothers after scheduled cesarean section (C-Section) births. Mothers that donate are fully informed, have healthy lifestyles, and are tested for infectious diseases.


PROKERA®




Healing on the Ocular Surface

What is amniotic membrane tissue?

Amniotic membrane is part of the placenta. It is the tissue closest to the baby throughout development in the womb. Amniotic membrane protects the baby from any harm and it has natural therapeutic actions which help the baby develop. This tissue has healing properties that aid in ocular surface repair.

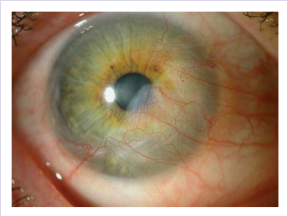

Chemical burn *before* PROKERA® treatment

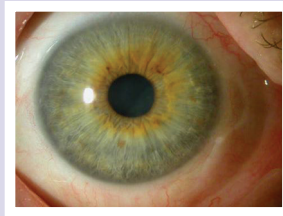

Chemical burn *after* PROKERA® treatment

What does PROKERA® do?

Helps the eye heal:


The amniotic membrane tissue in PROKERA® has natural therapeutic actions that help damaged eye surfaces heal. Eyes treated with PROKERA® have quicker healing, less scarring, and less inflammation and related pain.

Protects the eye surface:


The amniotic membrane in PROKERA® is thin and clear like the tissue on the surface of your eye and protects your eye's damaged tissue while inserted.

What does PROKERA® treat?

PROKERA® is used by eye doctors all over the world to treat eye diseases such as Keratitis, corneal scars, chemical burns, corneal defects, partial limbal stem cell deficiency and many other ocular surface diseases with inflammation.

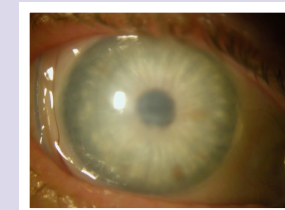

Corneal Scar *before* PROKERA® treatment

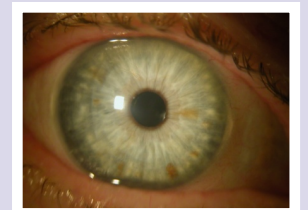

Corneal Scar *after* PROKERA® treatment

Is PROKERA® safe?

PROKERA® is a safe, effective treatment provided by a tissue bank regulated by the FDA. The tissue has passed numerous quality control tests before it is provided to your doctor. Ask your doctor if you have questions about using PROKERA® to treat your condition.

